



Echo All Terrain & Beach Wheelchair

IDENTIFY COMPONENTS TO START ASSEMBLY



- 1 - **BOTTOM SEAT SECTION WITH FOOTREST.**
- 2 - **ARMREST.**
- 4 - **BALLOON WHEELS.**
- 1 - **PUSH-PULL HANDLE.**
- 2 - **AXLE ROD**
- 4 - **COTTER PINS.**
- 1 - **MESH SLING SEAT FRAME.**
- 1 - **MESH BACKREST SLING.**
- 1 - **MESH ACCESSORY BAG.**
- 6 - **SCREWS TO SECURE ARMREST AND HANDLE.**

NOTE: PLIERS.

STEP 1 INSTALL SEAT SECTION.



- INSERT SLING SEAT FRAME INTO BOTTOM SEAT SECTION.**
- SECURE WITH CLAMP LOCK SYSTEM PROVIDED.**

STEP 2 INSTALL BALLOON WHEELS.



- PLACE FRAME WITH SEAT SECTION DOWN ON A FLAT SURFACE.**
- SLIDE AXLE ROD THROUGH BOTTOM SEAT FRAME.**
- INSERT WHEELS INTO AXLE.**

STEP 3 SECURE WHEELS.



- SECURE WHEELS WITH COTTER PINS BY PLACING PIN THROUGH THE HOLE AT THE END OF EACH AXLE.**



- BEND COTTER PIN TIP WITH PLIERS SO PIN DOES NOT FALL OUT.**

STEP 4 INSTALL ARMREST.



- INSERT ARMREST INTO BOTTOM SEAT FRAME.**
- SECURE ARMREST WITH SCREWS PROVIDED.**

Echo All Terrain & Beach Wheelchair

E720-ATC-KD-YEL

Assembly Instructions

STEP 5 INSTALL HANDLE.



- INSERT PUSH - PULL HANDLE INTO TOP OF ARMREST.
- SECURE HANDLE WITH SCREWS PROVIDED.

STEP 6 INSTALL MESH BACKREST SLING.



- PLACE BACKREST SLING ON FRAME, WEAVE THE WEBBING STRAP THROUGH THE PLASTIC LADDER LOCK FASTENER.
- ADJUST BY PULLING WEBBING STRAP.

STEP 7 FOLDING FOOTREST.



- SECURE FOOTREST WITH VINYL STRAP.

STEP 8.



- CHAIR COMPLETELY ASSEMBLED

NOTE: MESH SLING SEAT, MESH BACKREST SLING, ARMREST, HANDLE, AXLE ROD, COTTER PINS, MUST BE CHECKED AND SECURED PRIOR TO USING.

WARNING NOT SECURING ALL THE ABOVE CAN CREATE A SERIOUS SAFETY HAZARD.

Rev: 1.3